Operating Instructions

Japanese

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Dry & Store® Global 操作説明書

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Product Information

Physical Properties

- Outside dimensions: 180 mm x 99 mm x 91 mm (7.1" x 3.9" x 3.6"). Weight: 907 g (2 lb.).
- Drying tray has two compartments, a large drying area on the right which measures 102 mm x 90 mm x 25 mm (4" x 3.5" x 1"), and a small compartment on the left for the desiccant.

Power Supply

Input: 100-240 volts AC, 50/60 Hz, 1.0A maximum. Output: 24 VDC at 15W maximum.

Operating Features

- Unit turns off automatically after eight hours unless interrupted, but the normal cycle can be interrupted at any time by opening the lid. The germicidal lamp operates only during the first three minutes (see below).
- In just three minutes, the UV germicidal lamp delivers more than 1,000 joules/square meter, producing enough germicidal energy for a 99% kill on common microorganisms found in the external ear canal. Elimination of this flora from hearing aids can significantly reduce itching and the reintroduction of infection-causing bacteria into the ear canal. At 4 watts, the UV lamp is a surface sanitizer only, and it has little to no penetrating power. This means that it is not damaging to plastics or earmolds. The UV lamp has a life expectancy of many years, but it is user-replaceable if needed. (NOTE: Dry & Store is not designed to diagnose, prevent, monitor, treat, or alleviate disease.)
- Ball-bearing fan continuously circulates warm, dry air for the entire operating cycle, causing moisture to move out of the hearing instruments by diffusion. A desiccant in the airflow path strips the moisture out of the air, ensuring continuous dry operation.
- Dry & Store will remove the moisture from cerumen, decreasing its volume and making it easy to remove.

Desiccant

Dry & Store Global ships with a single disposable Dry-Brik® desiccant that also contains activated carbon for odor absorption. In most parts of the world, users will require 6 to 7 Dry-Briks per year, changing them once a month during warm weather and every two months the rest of the year. Dry-Briks may need to be changed monthly in tropical, sub-tropical, or other high-humidity locations. A rechargeable can desiccant is available as an accessory in many parts of the world.

Battery Removal

Batteries do not have to be removed from hearing aids before Dry & Store use. Independent tests by Eveready Battery Co. on their Energizer/Amplifier® zinc-air batteries showed a 10-20% gain in battery life under high humidity conditions. This was confirmed by a user survey which showed that over half of Dry & Store users noted longer zinc-air battery life.

Safety & Regulatory

Certified to ETL, UL, CSA standards. CE marked. PSE (Japan), C-Tick (Australia/New Zealand).

Dry & Store® is protected by U.S. Patents 5,640,783; 5,852,879; D 467-394;
Australian patent #732268; EPO, France & United Kingdom, #0980267;
Germany, #DE 697 28 438 T2; Spain, #ES 2218681 T3; Japan, #3705608.

Ear Technology Corporation
P.O. Box 1017
Johnson City, TN 37605-1017
USA
info@dryandstore.com
www.dryandstore.com