1

**Activate the Dry-Brik®** by removing the foil cover. Do not remove the foil cover until you are ready to use the Dry-Brik. Once the foil cover is removed, the Dry-Brik will begin adsorbing moisture.

2

**Write the date** that this Dry-Brik is placed in service.

Write the  
date here.

3

**Replace the Dry-Brik after 2 months.** Dispose of the old one in household refuse.

3

### Why use the Dry-Brik desiccant?

The Dry-Brik contained in the Dry & Store® family of products has two important functions: (1) to permanently remove the moisture molecules that are released from the devices during the warm drying cycle, and (2) to drive the relative humidity inside the box to a much lower level than is possible with just a heater, thus maximizing the drying capability.

It's important to change the Dry-Brik every two months in order to ensure that it has enough adsorptive capacity to capture the moisture released from your hearing instruments. Without a proper desiccant in place, the released moisture can just settle back on and in those instruments at the end of the 8-hour cycle.

Instrucciones de Funcionamiento  
Instructions d'Utilisation  
Bedienungsanleitung  
Gebruiksaanwijzing  
操作説明書・使用説明書

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